
Specimen Collection Guide

- Nasal Swabs

An anterior nasal swab sample can be collected by a medical professional or by an individual performing a self-swab.


Specimen collection, on children under 12 years of age, should be performed by a medical professional. Children aged 12 to 17 should be under adult supervision if they perform the anterior nasal swab by themselves. Adults aged 18 and over can perform the anterior nasal swab by themselves. Please follow your local guidelines for specimen collection by children.

MEDICAL PROFESSIONAL COLLECTION SELF COLLECTION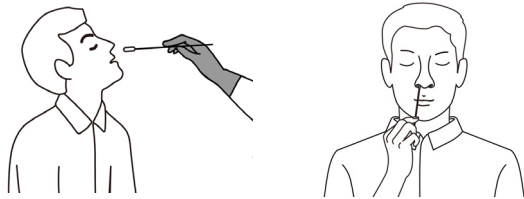

How to collect an anterior nasal swab sample:

- Carefully insert one of the Disposable Nasal Swabs, provided with your kit, into one nostril. Using gentle rotation, push the swab less than 2.5 cm (1 inch) from the edge of the nostril.
- Rotate the swab 5 times against the mucosa inside the nostril to ensure sufficient specimen collection.
- Using the same swab, repeat the process in the other nostril to ensure that an adequate amount of sample is collected from both nasal cavities.
- Withdraw the swab from the nasal cavity. The specimen is now ready for preparation using the extraction buffer tubes.

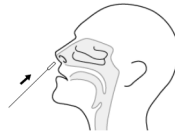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