# Conceive Plus

Individual applicators of vaginal lubricant formulated for use when you are trying to conceive.

## Conceive Plus™

- ✓ Safe for Sperm & Embryo development
- ✓ Electrolyte Balanced
- ✓ Contains Calcium & Magnesium Ions
- ✓ Supports Fertility Naturally

**Directions:** Remove one Conceive Plus™ individual applicator from the box, hold applicator narrow end facing upwards, completely twist off the end tab and discard. Gently insert the narrow end of the applicator deep into the vagina and squeeze the bulb end of the applicator 2-3 times to deposit Conceive Plus™ into the vaginal cavity. Remove the applicator while continuing to squeeze the bulb end and discard the used applicator. Do not re-use.

Apply 10 to 15 minutes prior to sexual activity to allow moisture to disperse throughout the vagina.

## Cautions

- Not a contraceptive, Conceive Plus™ does not kill sperm or interfere with sperm function.
- If irritation occurs, discontinue use.
- Keep out of reach of children
- Store between 2°C - 30°C
- Single use only, not safe for multiple use.
- Do not use if the tamper evident tab is broken or missing.

**Ingredients:** Water, Hypromellose, Sodium Chloride, Methylparaben, Glycerol, Sodium Phosphate, Potassium Chloride, Sodium dihydrogen Phosphate, Magnesium Chloride, Calcium Chloride.

**Further information** www.conceiveplus.com helpdesk@sasmar.com  
Tel +3228808220 Manufactured by SASMAR Sprl, 40-42 rue de l'Association B-1000, Brussels Belgium. This product is not a medicine.

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