THYROID-TSH TEST
TEST DE TIROIDES-TSH
TESTE DE TIROIDE-TSH

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Self-Test for the determination of the Thyroid Stimulating Hormone - TSH in blood

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Autoteste verificar a hormona tiroideia - TSH no sangue

THE TSH HORMONE AND THYROID

Thyroid disorders are one of the most prevalent medical conditions in the world, particularly for women, with an increasing rate of the population suffering under active thyroid (hypothyroidism). The symptoms of hypothyroidism include feeling tired and lethargic, weight gain, dry skin and brittle hair, becoming forgetful and sometimes depressed, feeling cold and enduring constipation as well as menstrual irregularities in women.

Thyroid stimulating hormone (TSH), which is secreted by the pituitary gland, regulates and controls the activity of the thyroid gland. Therefore, a raised TSH level indicates an under active thyroid and this is the principle on which the Thyroid Test is based for the detection of the hypothyroidism condition.

The **THYROID-TSH TEST** is a rapid immunodiagnostic test for the detection of increased TSH levels and therefore for hypothyroidism screening from a finger prick blood sample.

CONTENT

THE BOX CONTAINS ALL THE NECESSARY TO PERFORM 1 TSH TEST:

- 1 sealed aluminium pouch containing:

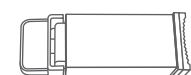
1 desiccant bag

1 test cassette

- 2 sterile lancets for blood sampling

- 1 disinfectant pad
- 1 instructions leaflet

Blue cap
White cap
- 1 dropper bottle containing 1 mL of diluent

- * Not provided necessary material: absorbent cotton.
- * Only open the protective pouch when you are ready to use the test.
- * The desiccant bag should not be used.

PRECAUTIONS

- Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
- Keep out of the reach of children.
- Do not use after the expiry date printed on the label and on the protective pouch or if the pouch is damaged.
- Follow strictly the indicated time, blood and diluent quantities.
- Store between +4 °C and +30 °C. Do not freeze.
- Do not re-use the cassette and lancet.
- External use only. DO NOT SWALLOW.
- This test is exclusively intended to in vitro diagnostic self-testing use.
- Not recommended for use by individuals taking medicines that thin the blood (anticoagulants) or by the individuals with disorders such as hemophilia.
- After use, dispose of all components in accordance with local regulations, seek advice from your pharmacist.

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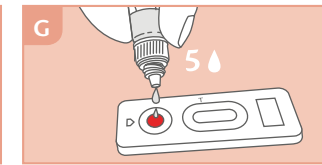
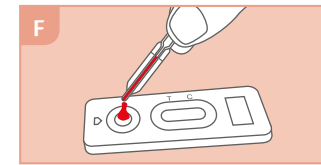
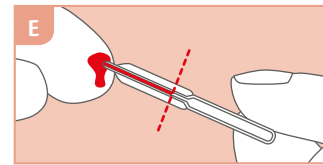
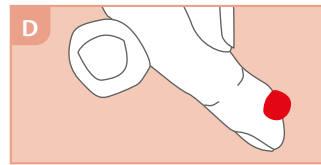
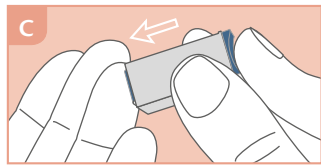
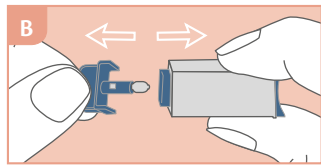
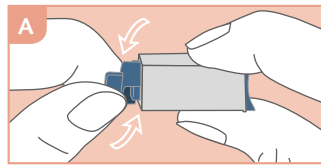
PROCEDURE

- 1) Wash your hands with soap and rinse with clear warm water.
- 2) Tear the protective pouch (from the notch) and only get out the cassette and the pipette. Dispose of the small desiccant bag.
- 3) Hold the lancet and carefully rotate 360° the protective cap without pulling it. - FIG. A
- 4) Extract and dispose the released cap. - FIG. B
- 5) Press the lancet, on the side from where the cap was extracted, against the fingertip (it is advisable the ring finger side). - FIG. C The tip retracts automatically and safely after use.
- 6) Keeping the hand down massage the end that was pricked to obtain a blood drop. - FIG. D
- 7) Without pressing the pipette bulb, put it in contact with the drop. The blood migrates into the pipette through the capillarity to the line indicated on the pipette. - FIG. E You may massage again your finger to obtain more blood if the line is not reached. As far as possible, avoid of air bubbles.
- 8) Put the blood collected with the pipette into the sample well of the cassette, by pressing on the pipette bulb. - FIG. F
- 9) Wait for the blood to be totally dispensed in the well. Unscrew the blue cap of the plastic dropper (leave the white cap tightly screwed) and add 5 drops of diluent into the sample well of the cassette. - FIG. G
- 10) Read the result after 10 minutes.

RESULT INTERPRETATION

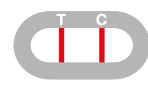
READ THE RESULT AFTER 10 MINUTES.
DO NOT READ THE RESULT AFTER 15 MINUTES.

The intensity of the line colour does not have any importance for the interpretation of the test result.


POSITIVE RESULT

Two coloured lines appear in the window under the marks T (Test) and C (Control). The intensity of the line T may be clearer than the intensity of the line C. This result means that the TSH level is higher than the norm and that you should consult a physician.


NEGATIVE RESULT

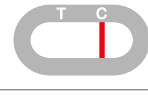Only one coloured line appears under the C mark (Control). This result means that the TSH level is normal.

NON VALID RESULT

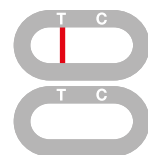No line appears or a coloured line appears under the mark T (Test) without any line under the mark C (Control). In this case, it is not possible to interpret the test, which must be considered as non valid. It is recommended to repeat the test with a new THYROID-TSH TEST and a new blood sample.

F.A.Q. - QUESTIONS & ANSWERS

HOW DOES THE THYROID-TSH TEST WORK? The thyroid stimulating hormone (TSH) is activating thyroid gland. Therefore a TSH level over 5µIU/mL in case of a positive result, indicates most probably an under active thyroid (hypothyroidism).

WHEN SHOULD THE TEST BE USED? In case of hypothyroidism symptoms such as feeling tired, depressed or cold regularly, weight gain, dry skin, brittle hair, enduring constipation or menstrual cycle irregularities in women, it is recommended to perform a THYROID-TSH TEST screening test. The THYROID-TSH TEST can be used any time of the day. However, it cannot and should not be performed in case of hormonal thyroid medical treatment.

CAN THE RESULT BE INCORRECT? The results are accurate as far as the instructions are carefully respected. Nevertheless, the result can be incorrect if the THYROID-TSH TEST gets wet before test performing or if the quantity of blood dispensed in the sample well is not sufficient, or if the number of diluent drops are less than 4 or more than 5. The plastic pipette provided in the box allows making sure the collected blood volume is correct.

HOW TO INTERPRET THE TEST IF THE COLOUR AND THE INTENSITY OF THE LINES ARE DIFFERENT? The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test line is.

WHAT IS THE LINE THAT APPEARS UNDER THE MARK C (CONTROL) FOR? When this line appears, it only means that the test was performing well.

IF I READ THE RESULT AFTER 15 MINUTES, WILL THE RESULT BE RELIABLE? No. The result should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

WHAT DO I HAVE TO DO IF THE RESULT IS POSITIVE? If the result is positive, it means that the TSH level in blood is higher than the norm (5µIU/mL) and that you should consult a physician to show the test result. Then, the physician will decide whether additional analysis should be performed.

WHAT DO I HAVE TO DO IF THE RESULT IS NEGATIVE? If the result is negative, it means that the TSH level is below 5µIU/mL and is within the norm. However, if the symptoms persist, it is recommended to consult a physician.

WHAT IS THE ACCURACY OF THYROID-TSH TEST? The THYROID-TSH TEST is very accurate and has been used for more than 10 years by professionals (hospitals, laboratories, ...) in the field. Evaluation reports show an overall agreement of at least 95% with reference methods.

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